## fabiani juhiste (FI) kaart

### ENNE PATSIENDI VENTILEERIMIST FABIANI VENTILAATORIGA

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## fabian Instructions (FI) Card

### **BEFORE VENTILATING A PATIENT WITH A FABIAN VENTILATOR:**

- **DO NOT** bypass flow sensor calibration process.
- **ALWAYS** follow the Instructions for Use.
- **ALWAYS** calibrate the O<sub>2</sub> sensor when it is manually enabled.
- **ALWAYS** power cycle the ventilator after changing languages.

### **WHILE VENTILATING A PATIENT WITH A FABIAN VENTILATOR:**

- **DO NOT** export trend data.
- **DO NOT** change the language setting.
- **DO NOT** use the Volume Limit function while in SIMV mode.
- **ALWAYS** follow the Instructions for Use.
- **ALWAYS** utilize external monitoring (e.g. SpO<sub>2</sub>, capnometry) with activated alarms.
- **ALWAYS** set patient alarms appropriately.
- **ALWAYS** set inspiratory pressure and high peak inspiratory pressure (PIP) alarm limit appropriately while taking into account that the pressure relief level is 12 mbar above set PIP.
- **ALWAYS** monitor adequacy of assisted ventilation in dual limb CPAP mode.

Keep this card visible close to/attached to fabian device until further notice!

If you have any questions, please contact your distributor