



Important Safety Warning / Customer Information

motion cycle 600 med / Welding Seam Failure

Dear Ladies and Gentlemen,

at least once, a welding seam at a 3 ½ years old motion cycle 600 med did brake, which lead to a complete separation of the monitor plate and the handles.

A review of production documentation shows that there were several lots of frame production, where the welding seams were traceably not conducted in the right size and quality.

Therefore there is a risk for the users, that the handles come of the ergometer during workouts at machines coming from these lots.

The serial numbers do indicate which machines might be affected by the problem described. The serial numbers contain the production date. The last four figures show month and year of production.

Trippstadter Straße 68, 67691 Hochspeyer
Gerätetyp: motion cycle 600 med
Serien-Nummer: **6MER016170917**
Norm: DIN EN ISO 20957-1 und DIN EN 957

Potentially affected are machines that end with 0815 to 0216 (August 2015 to February 2016)

We do recommend strongly to conduct the inspections listed in the user manual of the machines, which also contains a check of the visible welding seams. Until this has happened, the machines must be taken out of use.

At the same time, we do take corrective and preventive action to prevent machine failure for existing machines. We will equip the relevant machines with a adapter piece inside the machines monitor tube, which will reduce stress on the welding seam and at the same time prevents the handles to fall of also in case of a welding seam failure.

For questions around this topic, please do contact emotion fitness directly at:

emotion fitness GmbH & Co. KG
Trippstadter Str. 68
67691 Hochspeyer

info@emotion-fitness.de

TEL +49-6305-71499-0