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Monitor/defibrillaator Philips HeartStart MRx

Monitor/defibrillaator HeartStart MRx – ravi valimise lülit

KLIENDI/KASUTAJA RAKENDATAVAD MEETMED (jätkub)	Shift Checklist	20: Maintenance
	Shift Checklist	
	<p>In order to ensure defibrillators are ready when needed, the American Heart Association (AHA) recommends that users complete a checklist, often referred to as a shift check, at the beginning of each change in personnel. The activities on this check list include verifying that the appropriate supplies and accessories are present, the device is plugged in and has sufficient battery power, and the device is ready for use. Philips Healthcare supports the AHA checklist recommendations and has provided a Shift Checklist document with the device and published a copy in this book. See the "Appendix 1 - HeartStart MRx Shift Checklist" on page 363.</p>	
	<p>Weekly Shock Test</p> <p>In addition to the shift check, you must verify the ability to deliver defibrillation therapy once a week by performing <u>one of the following:</u></p> <ul style="list-style-type: none"> • Operational Check • Weekly Shock Test (See following instructions.) 	
	<p>NOTE: Test reusable sterilizable paddles (internal or external) prior to each use. See the <i>Sterilizable Defibrillator Paddles Instructions for Use</i> for more information.</p>	
	<p>◎ To perform the Weekly Shock test:</p> <ol style="list-style-type: none"> 1 If you are using paddles, make sure the paddles and the paddle tray are thoroughly clean and there is no debris or residue (including all conductive material) on the electrode surfaces of the paddles and tray. Secure the paddles in the paddle tray and confirm the Patient Contact Indicator (PCI) LEDs located on the sternum paddle are not lit. If the LEDs light, adjust the paddles in their pockets. If the LEDs continue to light, clean both the adult and pediatric paddle electrode surfaces. or If you are using multifunction electrode pads, attach a test load to the end of the patient Therapy cable. 2 Turn the Therapy knob to 150J. 3 Press the Charge button. 	
	<p>NOTE: If it becomes necessary to disarm the defibrillator, press [Disarm].</p> <ol style="list-style-type: none"> 4 The strip prints if configured to do so. If the strip does not print immediately, press the Print button. 5 If using: <ul style="list-style-type: none"> – Pads, press the Shock button on the MRx to deliver a shock into the test load. – External paddles, simultaneously press the shock buttons located on the paddles to deliver a shock into the pockets. 6 Confirm on the printed strip that the energy delivered to the test load is $150J \pm 23J$ (127J to 173J). If not, take the device out of use and begin troubleshooting. 	
	<p>NOTE: Detach the test load from the patient Therapy cable after performing the Shift Check. So your device is ready for use when needed, do not leave the test load attached after performing an Operational Check.</p>	
	<p>Teatise kättesaamise kinnitamiseks täitke kliendi vastusevorm ja saatke see järgmissele aadressile: service@meditsiinigrupp.ee</p>	

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