Medi-Globe GmbH, Medi-Globe-Str. 1-5, 83101 Rohrdorf / Achenmühle, Germany

**KUNDE**

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**GGW-03-XX-XXX; GGW-04-XX-XXX Bavarian Guidewire RU**

Dear Madams and Sirs,

With this letter we would like to inform you about a field corrective measure of our product **Bavarian Guide Wire RU** and ask for your attention:

**Identification of the affected medical devices:**

GGW-03-21-400	GGW-03-21-450	GGW-03-25-260	GGW-03-25-450	GGW-03-35-260	GGW-03-35-400
GGW-03-35-450	GGW-04-35-400	GGW-04-35-450	n/a	n/a	n/a

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**Description of the problem:**

We have been informed of a problem with the above product and its reprocessing, so that unrestricted, safe treatment of patients after reprocessing can no longer be guaranteed. It cannot be ruled out without a doubt that the material properties of the product are influenced by the reprocessing, e.g., the hydrophilic application.

Therefore, effective immediately, this product is no longer approved for multiple use and reprocessing.

We therefore declare the instructions for reprocessing invalid and hereby inform you not to follow the relevant passage in the instructions for use.

In the future, the Bavarian Guide Wires will no longer be offered as a reusable product and will only be available for single use.

**Patient risk & recommendation for further action:**

The safety and performance of the product after reprocessing cannot be fully guaranteed. Based on the current state of knowledge, we advise against reprocessing. However, single use is still allowed.

**Which measures are to be taken by the addressee?**

Upon tracing, we found that you received affected product.

For this reason, we ask you for the following support:

1. Please check whether you still have any of the above item numbers in stock.
2. Inform your end customers about the above-mentioned status change of the product.
3. Please refer to the attached customer information and include it with the products in your inventory.
4. Forward this Urgent Field Safety Notice to all those who need to be aware within your facility. If you have passed on the product as a dealer and/or member of a hospital association, please identify the affected facilities/departments and forward the safety and customer information to them immediately.
5. Please also inform your local authorities and monitoring authorities about this measure.
6. Please keep this information at least until the action has been completed.

The German Federal Institute for Drugs and Medical Devices has received a copy of this "urgent safety information".

7. Complete the attached response form and email it back to us immediately, but no later than March 15, 2023.

If you have any questions, we are always at your disposal.

**Your contact at MEDI-GLOBE:**

**FSCA-Gastro@medi-globe.de**

Best regards

Gerald Stegemann

---

**Gerald Stegemann/PRRC Medi-Globe GmbH**

## Response Form

Response requested at: [FSCA-Gastro@medi-globe.de](mailto:FSCA-Gastro@medi-globe.de)

**GGW-03-XX-XXX; GGW-04-XX-XXX Bavarian Guidewire RU**

We have the following number of affected products in stock:

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We have received the customer information and will take the necessary measures.

We have forwarded the safety information/customer information to the end customers.

We **do not have** any affected product in our inventory.

Sender:

---

---

---

Contact Person:

---

Phone No.

---

Comment:

---

---

Date:

Signature / Function

Please follow these instructions:

- 1. Fill out the reply form completely, even if you no longer have any products in stock.**
- 2. Please send the completed reply form to the following email address: [FSCA-Gastro@medi-globe.de](mailto:FSCA-Gastro@medi-globe.de)**

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