

Siemens Healthcare GmbH, SHS DI XP F&U PRM, Siemensstr. 3,  
Healthcare Gebäude 1, 91301 Forchheim

<To the person in charge of the unit where the SIEMENS  
product is operated, and the administrative head of  
organization>

Contact person of  
the Regional Unit  
Department

Telephone  
Fax  
Email

Date

### **Safety Advisory Notice**

To users of SIEMENS systems  
Luminos Agile Max, Axiom Luminos dRF, Luminos dRF  
Max, Ysio, Ysio Max, Ysio X.pree **with BABIX holder on  
the Bucky wall stand**

Contact person of  
the Business Unit  
Department

Monika Schwarz  
SHS DI XP F&U PRM

Telephone  
Fax  
E-Mail

+49 173 9655243

[monika.schwarz@siemens-healthineers.com](mailto:monika.schwarz@siemens-healthineers.com)

### **Re: Potential risk of falling of the BABIX holder on the Bucky wall stand**

Dear customer,

This letter is to inform you about a potential problem and risk to persons that could occur when using the BABIX holder on the Bucky wall stand.

#### **When could the hazard occur and what are the potential risks?**

Following a complaint, it was identified that during system installation the mounting of the BABIX holder was possibly performed incorrectly. This results in a potential risk that during attachment of the BABIX cradle, the BABIX holder might become loose and fall down, even if it has previously been used without any problems.

This could lead to injuries of the operator and in worst case might be fatal for examined infants. No serious incidents are known. In the present case there was a minor injury to the operator.

**Siemens Healthcare GmbH**  
Managing Board: Bernhard Montag, Chairman and Chief Executive Officer;  
Darleen Caron, Jochen Schmitz

Chairman of the Supervisory Board: Ralf P. Thomas  
Registered offices: Munich, Germany; Commercial registry: Munich, HRB 213821  
WEEE Reg. No. DE 64872105

Siemensstr. 3  
91301 Forchheim  
Germany

Tel.: +49 (9191) 180  
[siemens.com/healthcare](http://siemens.com/healthcare)

**What steps can the user take to avoid the potential risk of this issue?**

The BABIX holder may not be used until a service engineer has checked the correct mounting onsite.

**How will the issue finally be resolved, and the corrective action be implemented?**

This advisory letter (Update XP056/23/S) is distributed to all potentially affected customers.

The field corrective action (Update XP047/23/S) will initiate an onsite visit of a service engineer to check the mounting for the BABIX holder on the Bucky wall stand and if necessary, mount it correctly. Update XP047/23/S is planned to be released in October 2023 and will be provided free of charge.

Our service organization will contact you for an appointment to perform the corrective action. Please feel free to reach out to the service organization for an earlier appointment.

We appreciate your understanding and cooperation with this safety advisory notice and ask you to immediately inform your personnel accordingly. Please ensure this safety advisory notice is maintained in the system's instructions for use.

If the device has been sold and is therefore no longer in your possession, please forward this safety notice to the new owner. Please inform us about the new owner of the device.

Sincerely yours,

Verena Schön  
Head of Business Line  
X-Ray Products

Andreas Herdegen  
Head of Quality Management  
X-Ray Products

**Signature:**



*Electronically signed by: Verena  
Schoen  
Reason: I am approving this document  
Date: Sep 11, 2023 15:35 GMT+2*

**Email:** verena.schoen@siemens-healthineers.com

**Signature:**



*Electronically signed by: Andreas  
Herdegen  
Reason: I am approving this document  
Date: Sep 11, 2023 15:41 GMT+2*

**Email:** andreas.herdegen@siemens-healthineers.com